

BioCLIA Allergy Reagent Kit, Sx1

(Chemiluminescent Microparticle Immunoassay)

REF

MY01308 (25 Tests/kit)

MY01306 (50 Tests/kit)

MY01304 (100 Tests/kit)

INTENDED USE

BioCLIA Allergy Reagent Kit, Sx1 is an *in vitro* qualitative assay for the determination of mixed specific IgE to House dust mite (*Dermatophagoides pteronyssinus*, D1), Cat dander (E1), Dog dander (E5), Rye-grass(G5), Timothy grass (G6), *Alternaria alternata* (M6), Common silver birch (T3), Mugwort (W6) in human serum via chemiluminescent immunoassay method on the fully automatic BioCLIA instrument. It is intended for *in vitro* diagnostic use as an aid in the clinical diagnosis of IgE mediated allergic disorders specific to D1, E1, E5, G5, G6, M6, T3, W6 in conjunction with other clinical findings, and is to be used in clinical laboratories.

SUMMARY AND EXPLANATION OF THE TEST

Allergies, also known as allergic diseases, are a number of conditions caused by hypersensitivity of the immune system to something in the environment that usually causes little or no problem in most people. These diseases include hay fever, food allergies, atopic dermatitis, allergic asthma, and anaphylaxis. Symptoms may include red eyes, an itchy rash, runny nose, shortness of breath, or swelling.¹

Allergy development results from both genetic and environmental factors. The underlying mechanism involves immunoglobulin E antibodies (IgE), part of the body's immune system, binding to an allergen and then a receptor on mast cells or basophils where it triggers the release of inflammatory chemicals such as histamine.²

Allergies are common. In the developed world, about 20% of people are affected by allergic rhinitis,³ about 6% of people have at least one food allergy,⁴ and about 20% have atopic dermatitis at some point in time.⁵ Depending on the country, about 1–18% of people have asthma.⁶ Anaphylaxis occurs in 0.05–2% of people. Incidence of many allergic diseases appears to be increasing.⁷

PRINCIPLE OF THE PROCEDURE

BioCLIA Allergy Reagent Kit is a two-step immunoassay. The allergen of interest is biotinylated, it reacts with the specific IgE in patient sample and form a IgE-allergen complex with streptavidin beads. After incubation, a washing step removes the unbound reagents and sample matrix. Subsequently add enzyme conjugated anti-human IgE antibodies and bind to the IgE-allergen complex. After incubation, a second washing step removes the excess enzyme conjugates. Then addition of substrate results in the emission of light and RLU is measured. RLU is proportional to the amount of allergen specific IgE present in the sample. To evaluate the test results, RLU is transformed to concentration with the use of a calibration curve.

WARNINGS AND PRECAUTIONS

- For professional *in vitro* diagnostic use only.
- Used on BioCLIA 6500 and BioCLIA 500 instruments only.
- Do not use reagents beyond the expiration dates.
- The kit contains human sourced materials. All recommended precautions for the handling of blood derivatives should be taken. Please refer to the OSHA Standard on Blood borne

Pathogens⁸ or Biosafety Level¹⁰ or other appropriate biosafety practices.^{10,11}

- Liquid waste and solid waste taken from BioCLIA 6500 and BioCLIA 500 should also be handled in accordance with the appropriate biosafety practices same as stated above.
- Once opened, the reagent cartridge must be stored in the instrument's reagent carousel. For the first placement of reagent into the instrument, please take care to avoid spilling the reagents.
- Spilled reagents should be cleaned up immediately. Comply with all federal, state and local environmental regulations when disposing of wastes.

Improper cleaning or rinsing of the instrument may lead to chemical contamination of the reagents. Residues from common laboratory chemicals such as formalin, bleach, ethanol, or detergent can cause interference in the assay. Be sure to follow the recommended cleaning procedure as outlined in the BioCLIA 6500 and BioCLIA 500 User's Manual.



Precautions

- The allergen contains ProClin300 0.1% as preservative and Triton X-100 0.01% as surfactant and may cause an allergic skin reaction by skin contact. Avoid contact with skin. Wear protective gloves, protective clothing and take eye protection.
- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

MATERIALS SUPPLIED

- Sx1 Allergen** Mixed biotinylated D1, E1, E5, G5, G6, M6, T3, W6 antigen in 0.01 M PBS (pH7.4 ± 0.2) buffer with stabilizer.

Sx1	BIOT
-----	------

Preservatives: 0.1% ProClin300;

Surfactant: 0.01% Triton X-100.

Kit size	Vial Quantity	Volume/Vial (mL)
25 Tests/Kit	1	0.375
50 Tests/Kit	1	0.75
100 Tests/Kit	2	0.75

ADDITIONAL MATERIALS SUPPLIED SEPARATELY

Product	CATALOGUE No.
BioCLIA Allergy Specific IgE Conjugate & Microparticle Set	MY01311 (100 Tests/Kit) MY01312 (150 Tests/Kit) MY01313 (200 Tests/Kit) MY01314 (300 Tests/Kit)
BioCLIA Allergy Specific IgE Calibrator and Control Set	MY01315
BioCLIA System Wash Buffer	MY00404
BioCLIA System Substrate	MY00405
BioCLIA 6500	MA00243
BioCLIA 500	MA00502
BioCLIA Cuvettes	MA00244 (2000 pcs/bag) MA00549 (65 pcs/box)
BioCLIA Silicone gasket (Small)	MV00195

BioCLIA Silicone gasket (Large)	MV00196
---------------------------------	---------

MATERIALS REQUIRED

- Distilled or deionized Water

STORAGE AND STABILITY

- Store the kit at 2-8 °C.
- The shelf life of the unopened kit is 18 months from day of production.
- Vial opened reagents or onboard reagents could be used for 28 successive days. The software of the BioCLIA instruments monitors the onboard (in-use) expiration of the reagent cartridge. The system will not allow use of a reagent which has expired.

SPECIMEN COLLECTION, STORAGE AND HANDLING

- Serum from venous can be used.
- Collect blood specimens using standard procedures.
- Test serum should be clear and non-hemolysis.
- Cloudy samples should be clarified by 3000r/min for 10 minutes before use. For samples with the presence of fibrin, ensure that complete clot formation has taken place prior to centrifugation of samples. Some samples, particularly those from patients receiving anticoagulant therapy, may require increased clotting time.
- Specimens may be refrigerated at 2-8 °C for up to seven days or stored at -20°C up to 36 months.
- Specimens may be kept onboard on BioCLIA instruments under room temperature (18-25°C) for up to 8 hours.
- Avoid repeated freezing and thawing.

ASSAY PROCEDURE

Detailed information about operating the BioCLIA instruments can be taken from the Instrument User's Manual.

Note that, it is important to perform all routine maintenance procedures for optimal performance.

Assay Calibration

The BioCLIA Allergy Reagent Kit utilizes seven points of calibrators to get the Working Calibration Curve. The calibrators information is uploaded into the instrument via the barcode provided in the BioCLIA Allergy Specific IgE Calibrator and Control Set. The Working Calibration Curve is generated and is used to calculate the analyte concentration (IU/mL) from the RLU obtained for each patient.

For each new lot of reagent, please calibrate prior to the first time use, and every 28 days thereafter. The software will not allow the lot to be used if the above requirements are not meet.

Programming and Running Samples

1. Put the reagents into the corresponding position of the reagent chamber of the fully automatic chemiluminescence analyzer. The information of the reagents can be uploaded into the instrument system through the scanning of reagent barcode, and can also be set through the supporting software of the instrument.
2. The information of calibrator / quality control is identified by scanning the calibrator / control barcodes, and the position of calibrator / quality control is assigned in the instrument system.
3. The sample to be tested is placed on the instrument sample rack chamber, and the corresponding test information is edited

through the instrument supporting software.

4. Start the operation procedure, and all calibrator / quality control / sample processing steps will be automatically executed.

Control

The control procedure could be done before running the specimens every day. Users also can adjust the control procedure period according to their own lab frequency.

CALCULATION OF RESULTS

Calculation and interpretation of results will be performed automatically by software on BioCLIA instruments.

RESULT INTERPRETATION

Classification System (Scoring):

Class	Conc. (IU/mL)	Interpretation
0	0-0.34	Negative/Equivocal
1	0.35-0.69	Low
2	0.70-3.49	Moderate
3	3.50-17.49	High
4	17.50-49.99	Very High
5	50.00-99.99	Very High
6	≥ 100	Very High

The test results only reflect the amount of allergen specific IgE presented in the collected sample and should be diagnosed in conjunction with other laboratory and clinical findings.

CUT-OFF VALUE DETERMINATION

Clinical significance cut-off of allergen-specific IgE concentration has been set at 0.35 IU/mL for indication for sensitization. BioCLIA allergy reagent kits have been set using this as the cut-off to differentiate positive verses negative results.

PERFORMANCE CHARACTERISTICS**APPEARANCE**

Kit components are complete with no liquid leakage, no precipitation or floc in liquid reagents. Packing labels should be clear and easy to identify.

TRACEABILITY

The IgE calibrators can be traced to WHO International Standard Immunoglobulin E (IgE), human serum (NIBSC code: 11/234).

PRECISION

A study based on guidance from (NCCLS) document EP-A¹⁸ was performed.

Intra-assay precision: Four samples (negative, low, moderate, and high) were taken and tested with 10 replicates for each in a single run. Coefficient of variation (CV) was calculated for each of four samples. The results for intra-assay precision are shown in the table below.

Inter-assay precision: Four samples (negative, low, moderate, and high) were taken and tested with 4 replicates in a single run, two runs per day for 10 days. Coefficient of variation (CV) was calculated for each of four samples. The results for inter-assay precision are shown in the table below.*

Intra-assay precision: CV ≤ 10%

Intra-Assay	negative	low	mid	high
Mean	0.31	0.84	16.80	80.75
CV	7.0%	1.7%	3.9%	1.7%

Inter-assay precision: CV ≤ 15%

Inter-Assay	negative	low	mid	high
Mean	0.28	0.76	18.42	83.58
CV	8.8%	6.3%	7.8%	3.8%

*Representative data; results in individual laboratories may vary from these data.

LIMIT OF BLANK / DETECTION (LoB/LoD)

LOB/LOD was determined according to CLSI EP17-A guideline.

The assay is designed to have LoB/LoD of ≤ 0.35 IU/mL.

INTERFERENCE

No interference has been observed with bilirubin, hemoglobin, triglycerides, rheumatoid factor (RF), human anti-mouse antibody (HAMA) at the level indicated below: *

- Bilirubin ≤ 200 mg/dL;
- Hemoglobin ≤ 400 mg/dL;
- Triglycerides ≤ 2,000 mg/dL;
- Rheumatoid factor (RF) ≤ 1,000 IU/mL;
- Human anti-mouse antibody (HAMA) ≤ 2,000 ng/mL.

CROSS REACTIVITY

One allergen specific IgE blank sample was spiked with the IgA/IgG/IgM to a different concentration and tested with the assay. No false positive results are observed for the samples spiked with IgA/IgG/IgM at 700,7000,500µg/mL, respectively.

METHOD COMPARISON

246 samples collected in three different hospitals were from individuals suspected of allergy and from healthy controls. The patients were diagnosed according to each investigator's routine procedures as either atopic or non-atopic prior to the assay. The results are shown in the table below. *

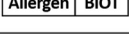
	Study 1	Study 2	Study 3
Sensitivity	94.44%	96.77%	95.2%
Specificity	91%	96%	92%
Efficiency	93%	96.7%	94.6%

*Representative data; results in individual laboratories may vary from these data.

LIMITATIONS

- The effectiveness of this kit is only confirmed for human sera, the applicability of the other kinds of samples is not verified.
- Reliable and reproducible results will be obtained when the assay procedure is carried out in accordance with the instructions and with adherence to good laboratory practice.
- Clinical diagnosis should not be made on the findings of a single test result, but should be interpreted with all clinical and laboratory findings.

SYMBOLS

	Catalog Number		Use-by date
	In Vitro diagnostic medical device		Lot Number
	Store between +2°C and +8°C		Consult Instruction for Use
	Manufacturer		Authorized Representative in the European Community
	CE Marking		Contains Sufficient for <n> Tests
	Biological Risk		GHS07 Warning
	Biotinylated allergen		

REFERENCES

- McConnell TH. The nature of disease: pathology for the health professions: Lippincott Williams & Wilkins, 2007.
- Kay AB. Overview of 'allergy and allergic diseases: with a view to the future'. British Medical Bulletin 2000;56:843-64.
- Wheatley LM, Togias A. Clinical practice. Allergic rhinitis. New England Journal of Medicine 2015;372:456-63.
- Sicherer SH, Sampson HA. Food allergy: epidemiology, pathogenesis, diagnosis, and treatment. J Allergy Clin Immunol. 2014;133:291-307.
- Portnoy JM, Amado M. Evidence-based allergy diagnostic tests. Current Allergy and Asthma Reports 2006;6:455-61.
- Li JT, Andrist D, Bamlet WR, Wolter TD. Accuracy of patient prediction of allergy skin test results. Annals of Allergy Asthma & Immunology Official Publication of the American College of Allergy Asthma & Immunology 2000;85:382-4.
- Yunginger JW, Ahlstedt S, Eggleston PA, Homburger HA, Nelson HS, Ownby DR, et al. Qualitative IgE antibody assays in allergic diseases. Journal of Allergy & Clinical Immunology 2000;105:1077-84.
- US Department of Labor, Occupational Safety and Health Administration, 29 CFR Part 1910.1030, Occupational Exposure to Bloodborne Pathogens.
- US Department of Health and Human Services. Biosafety in Microbiological and Biomedical Laboratories, Fourth Edition. Washington, DC: US Government Printing Office, May 1999.
- World Health Organization. Laboratory Biosafety Manual. Geneva: World Health Organization; 2004.
- Clinical and Laboratory Standards Institute. Protection of Laboratory Workers from Occupationally Acquired Infections: Approved Guideline - Third Edition. CLSI Document M29-A3. Wayne, PA: Clinical and Laboratory Standards Institute, 2005.



HOB Biotech Group Corp., Ltd.
C6 Building, No. 218 Xinghu Road, Suzhou Industrial Park,
Suzhou, Jiangsu, 215123, China

CONTACT INFORMATION:

TEL (+86)512-69561996

Fax (+86)512-62956652

WEBSITE: www.hob-biotech.com

CUSTOMER SERVICE: TEL (+86)4008601202



EUROPE REPRESENTATIVE: Emergo Europe

ADDRESS/LOCATION:

Prinsessegracht 20, 2514 AP The Hague, The Netherlands



The eIFU is available on Website:

<http://en.hob-biotech.com/usercenter/login.aspx>

TECHNICAL ASSISTANCE

For technical assistance, contact your National Distributor.

Date of issue: 15th Nov. 2021

Change Control Number: CN19056E

Version: A/0 (EN)